



SANITIZATION PROTOCOL FOR OXYGEN CONCENTRATOR

- CLEAN & SANITIZE OXYGEN CONCENTRATOR BEFORE & AFTER USE FOR EVERY PATIENT.
- DISCARD USED CANNULA. USE BRAND NEW CANNULA FOR YOUR PATIENT.
- Make sure concentrator is switched OFF & Unplugged.
- Dip clean cloth in warm soap (dishwash soap) solution, squeeze it damp & wipe the concentrator.
- Dip clean cloth in water, squeeze it damp & wipe the concentrator & let it air-dry.
- Wipe concentrator with cloth dipped in alcohol-based sanitizer solution & let it air dry.
- **To wash humidifier bottle** – soak bottle in warm soap solution >> Rinse with vinegar + water solution (1:10) solution >> let bottle dry completely.
- If returning the concentrator, pack it securely after cleaning & sanitizing & keep ready for dispatch.
- DO NOT pack cannula used by patient along with concentrator. DISCARD it.